


INSTRUCTION SHEET – For manual TIP cleaning of ERCP12 series endoscopes

Warning 1: Only use **pH neutral** cleaning detergent, such as “Flexinc Powerclean 1200”.

Warning 2: Only perform cleaning routine if endoscope has passed leakage test!

#	Action
1	Clean submerged endoscope with clean soft cloth
2	Brush tip all around and according to (T) – each action 3 times
3	Move submerged forceps elevator all the way up and all the way down – 3 times
4	Brush working channels according to (C) – 3 times
5	Flush working channel using 50 ml syringe with cleaning liquid – 2 times
6	Move submerged forceps elevator all the way up and all the way down – 3 times
7	Flush forceps elevator using 50 ml syringe with cleaning liquid – 2 times with elevator up, 2 times with elevator down until visually clean

(T) Brushing the forceps elevator area

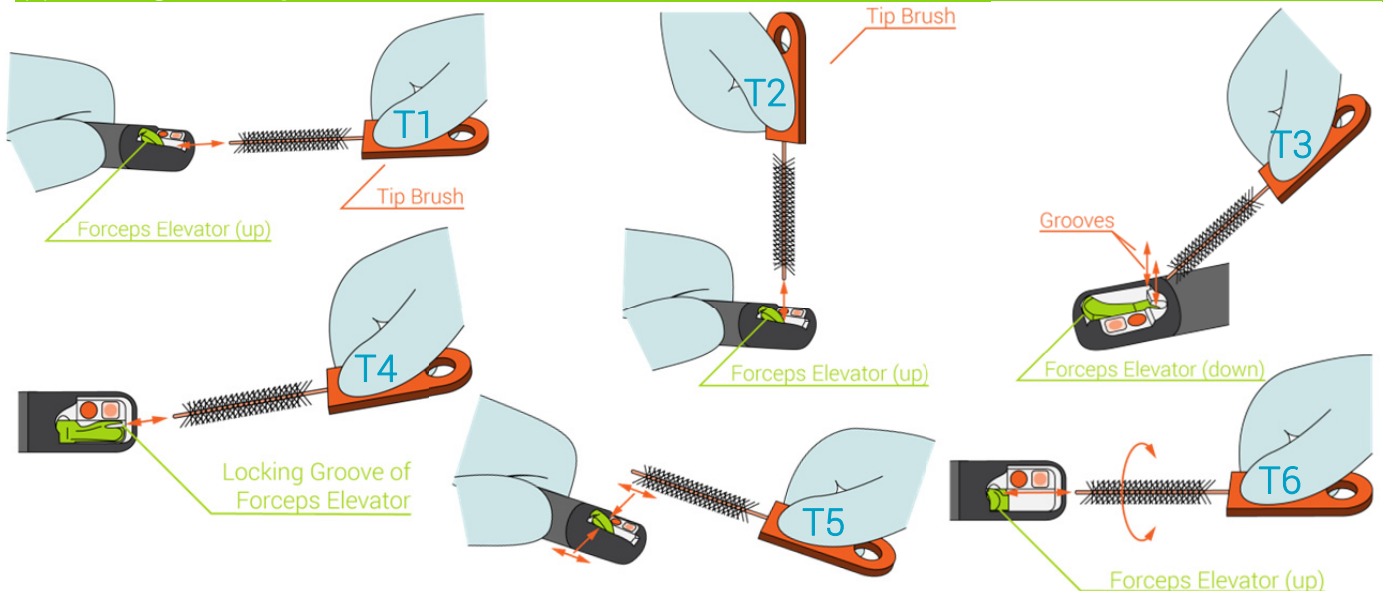

(C) Brushing the working channel

